

# How to measure for Tubbease


Simply measure the widest diameter of your horse's hoof sole (this might be length or width depending on which is longer) and fit *within* one of the Tubbease sizes below. The fit of the hoof sock must be as close as possible to the hoof.

Tubbease can also be fitted over horseshoes.

**Please note:** The Tubbease dimensions shown below indicate the *maximum internal diameter* of the Tubbease cup *not* the diameter of the hoof.




## Apply your Tubbease in four easy steps:

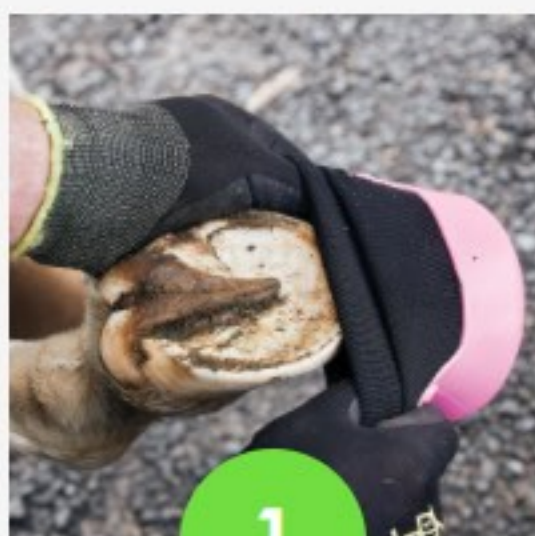


### Pull over hoof

After preparing the hoof, open sock wide and pull Tubbease over hoof using tabs at side.

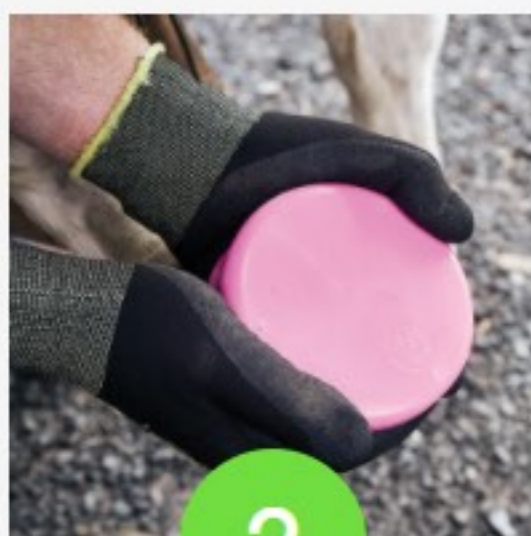


### Check fitting

Check Tubbease is fitting hoof correctly, then pull sock up carefully.

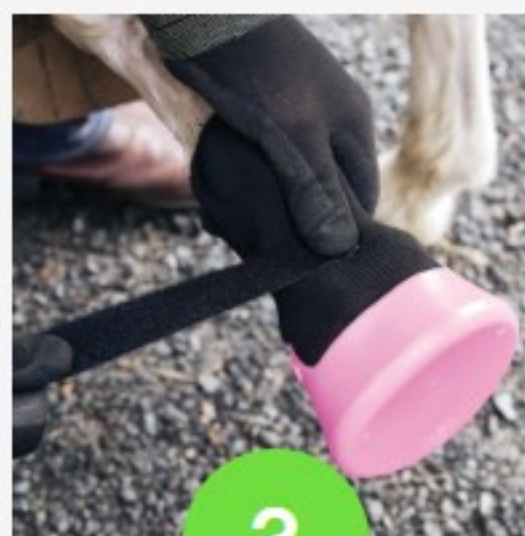

### Apply strap

With sock taut, fit hook and loop strap to lower pastern with hook side facing sock. Ensure one finger space under strap.

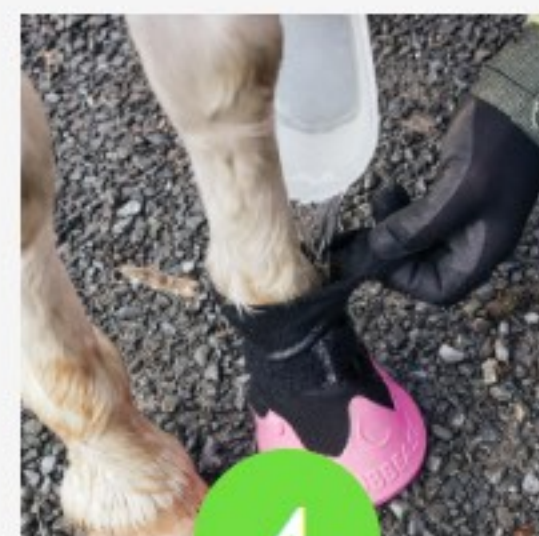

### Add treatment solution

With hoof on the ground, simply pour in treatment solution as required, or submerge fitted hoof into a bucket of your chosen solution for deeper soaking.

**Important:** Tubbease Hoof Socks are not enduring boots. They are for short-term use in confined enclosures by restrained animals. Judgement should be exercised.





Mini  
75mm/3"



Small  
110mm/4.5"



Medium  
145mm/5.75"



Large  
155mm/6.125"



X-Large  
165mm/6.5"

## Sole Inserts



Mini  
75mm/3"



Small  
110mm/4.5"



Medium  
145mm/5.75"



Large  
155mm/6.125"



X-Large  
165mm/6.5"