

For professional use only. Keep out of reach of children.
Do not burn or hot shoe this product.

To open, cut off tip (at line) with nippers or knife. Insert cartridge into dispenser gun with label side up. Squeeze trigger until push rods engage cartridge. Squeeze out small amount of Equi-Pak™ to equalize components. Attach mixer tip. Dispense.


Limited Remedy: Except for replacement or refund remedies, manufacturer is not liable for direct damages or liable for indirect, incidental or consequential damages regardless of legal theory asserted, including assertion of negligence and strict liability.

CONTAINS: castor oil, polymer with 1,1'-methylenebis[4-isocyanatocyclohexane] (68459-77-8); oxirane, methyl-, polymer with oxirane, ether with 1,2,3-propanetriol (3:1), polymer with 1,1'-methylenebis[4-isocyanatocyclohexane] (67837-35-8); 2-propanol, 1,1',1'',1'''-(1,2-ethanediyldinitrilo)tetrakis- (102-60-3); benzene, 1,1'-methylenebis [isocyanato-, homopolymer (39310-05-9); methylenediphenyl diisocyanate (26447-40-5); neodecanoic acid, bismuth(3+) salt (34364-26-6); 1,4-butanediol (110-63-4); and decanedioic acid, bis(1,2,2,6,6-pentamethyl-4-piperidiny) ester (41556-26-7)

SUGGESTED FIRST AID: Eye Contact: Flush eyes with large amounts of water. If signs/symptoms persist, get medical attention. **Skin Contact:** Remove contaminated clothing. Immediately flush skin with large amounts of water. Get medical attention if irritation develops. **Ingestion:** Drink two glasses of water. Do not induce vomiting. Get medical attention.

U.S. Patent No. 6,009,952; 6,021,851; 6,131,665; 6,196,326;
6,412, 566; 6,364,025
Other U.S. and Foreign Patents Pending. Made in U.S.A.

REF 46121

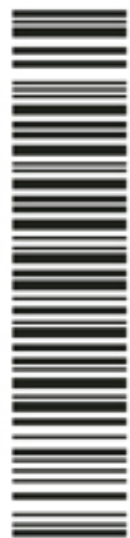

+D696461212

Equi-Pak™

Download complete product
instructions at www.vettec.com

Vettec

600 East Hueneme Road
Oxnard, CA 93033 USA
800.483.8832
www.vettec.com

Zonnebaan 14
NL-3542 EC Utrecht
The Netherlands
+(31) 30 241 1823

Lot No:

LP46121-03

Equi-Pak™

Soft Instant Pad Material

Clear

Net Contents
180cc / 6oz

30 sec. set time

46121


Vettec
HOOF CARE PRODUCTS

www.vettec.com