This product should not be used in the presence of bleeding or white necrotic tissue. For professional use only. Keep out of reach of children. Do not burn or hot shoe this product.

To open, cut off tip (at line) with nippers or knife. Insert cartridge into dispenser gun with label side up. Squeeze trigger until push rods engage cartridge. Squeeze out small amount of Equi-Pak|CS™ to equalize components. Attach mixer tip. Dispense.

Limited Remedy: Except for replacement or refund remedies, manufacturer is not liable for direct damages or liable for indirect, incidental or consequential damages regardless of legal theory asserted, including assertion of negligence and strict liability.

CONTAINS: castor oil, polymer with 1,1'-methylenebis[4-isocyanatocyclohexane] (68459-77-8); oxirane, methyl-, polymer with oxirane, ether with 1,2,3-propanetriol (3:1), polymer with 1,1'-methylenebis[4-isocyanatocyclohexane] (67837-35-8); 2-propanol, 1,1',1"',1"''-(1,2-ethanediyldinitrilo)tetrakis- (102-60-3); benzene, 1,1'-methylenebis [isocyanato-,homopolymer (39310-05-9); methylenediphenyl diisocyanate (26447-40-5); neodecanoic acid, bismuth(3+) salt (34364-26-6); and copper(ii)sulfate, pentahydrate (7758-99-8).

SUGGESTED FIRST AID: Eye Contact: Flush eyes with large amounts of water. If signs/ symptoms persist, get medical attention. Skin Contact: Remove contaminated clothing. Immediately flush skin with large amounts of water. Get medical attention if irritation develops. Ingestion: Drink two glasses of water. Do not induce vomiting. Get medical attention.

U.S. Patent No. 6,009,952; 6,021,851; 6,131,665; 6,196,326; 6,412, 566; 6,364,025 Other U.S. and Foreign Patents Pending, Made in U.S.A.



## Equi-Pak CS

Download complete product instructions at www.vettec.com

## Vettec

600 East Hueneme Road Oxnard, CA 93033 USA 800.483.8832 www.vettec.com

Zonnebaan 14 NL-3542 EC Utrecht The Netherlands +(31) 30 241 1823

Lot No:



## Soft Instant Pad Material for Thrush

Green

Net Contents

1 minute set time

46122



www.vettec.com

