

BEFORE YOU DEWORM *ParaScreen-e!*



Every horse's routine health care program needs to include regular fecal testing. Pre- and post-treatment fecal testing is the best way for horse owners to establish and monitor effective deworming programs and control common equine intestinal parasites.

New England Horse Labs, a USDA-APHIS inspected and certified equine testing lab, now makes it easy for horse owners to obtain fecal testing. Get fast, accurate results with **ParaScreen-e™**.

The **ParaScreen-e™** kit includes the cost for testing and everything you need for collecting and mailing an equine fecal sample:

- One **ParaScreen-e™** fecal submission form
- Easy to follow instructions
- One specimen container, scoop and glove
- One pre-paid, pre-addressed mailing envelope

Testing will be completed within 24 hours after receipt of your sample. Results will be sent directly to you via email or US Mail. You will receive a "Fecal Analysis" reporting a "McMaster's Quantitative" fecal egg count, a valuable tool for establishing and monitoring the effectiveness of your deworming program.

Instructions

1. Remove the specimen container and plastic scoop from the enclosed zip-lock bag.
2. Clearly print your horse's name on the label of the zip-lock bag.
3. Using the plastic scoop or your gloved hand, collect enough FRESH fecal sample to completely fill the black container.
4. Close the lid of the black container and place it back in the labeled zip-lock bag. Seal the bag. Refrigerate the sample if you are not going to mail it the same day.
5. Complete the Fecal Submission Form. Please print clearly.
6. Insert the completed form and fecal sample into the postage-paid, pre-addressed specimen envelope, seal it and mail it to:

New England Horse Labs
7 Lenora Street
Worcester, MA 01607

All **ParaScreen-e™** test results should be discussed with your veterinarian to determine the most effective treatment options. Please do not call the lab to inquire about treating your horse. New England Horse Labs is a testing facility only.

Attention Email Recipients: If you do not receive your test results from New England Horse Labs within one week, please check your spam mail before contacting the lab.

parascreen.com