


The Easyboot Trail Original Size Charts

Easyboot Trail - Inches		
Size	Width in inches	Length in inches
0	3 15/16" - 4 1/8"	4 1/8" - 4 5/16"
1	4 1/8" - 4 5/16"	4 5/16" - 4 1/2"
2	4 5/16" - 4 1/2"	4 1/2" - 4 3/4"
3	4 1/2" - 4 3/4"	4 3/4" - 4 15/16"
4	4 3/4" - 4 15/16"	4 15/16" - 5 1/8"
5	4 15/16" - 5 1/8"	5 1/8" - 5 5/16"
6	5 1/8" - 5 5/16"	5 5/16" - 5 1/2"
7	5 5/16" - 5 1/2"	5 1/2" - 5 11/16"
8	5 1/2" - 5 11/16"	5 11/16" - 5 15/16"
9	5 11/16" - 5 15/16"	5 15/16" - 6 1/8"
10	5 15/16" - 6 1/8"	6 1/8" - 6 5/16"
11	6 1/8" - 6 5/16"	6 5/16" - 6 11/16"
12	6 1/2" - 6 7/8"	6 11/16" - 7 1/8"

Easyboot Trail - MM		
Size	Width in mm	Length in mm
0	100-105mm	105-110mm
1	105-110mm	110-115mm
2	110-115mm	115-120mm
3	115-120mm	120-125mm
4	120-125mm	125-130mm
5	125-130mm	130-135mm
6	130-135mm	135-140mm
7	135-140mm	140-145mm
8	140-145mm	145-150mm
9	145-150mm	150-155mm
10	150-155mm	155-160mm
11	155-165mm	160-170mm
12	165-175mm	170-180mm

How to Properly Measure Your Horse's Hoof

1. After a fresh trim, measure the width of the hoof across the bottom at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurement with the appropriate size chart. Please note that each boot style has its own unique size chart.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, your selected boot style is not recommended.* Be sure to check your measurements against an alternative boot in the EasyCare line—another style may accommodate your horse's hoof shape better.

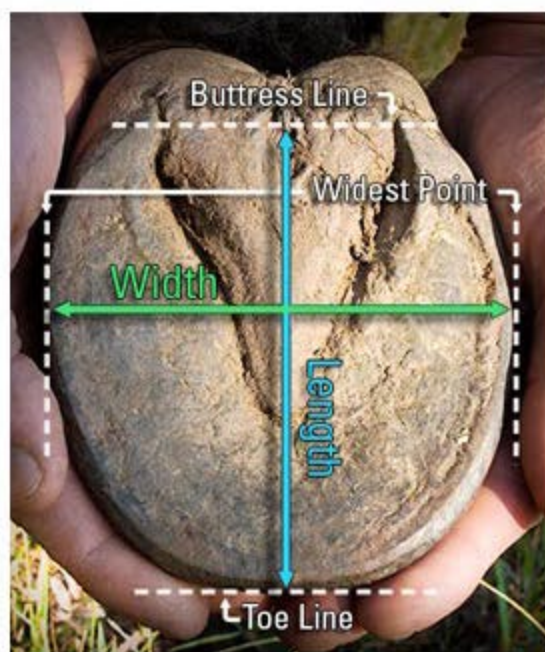

*Note: If you are fitting an Easyboot Glue-On or Glove, width and length must fall into the same size category.