

EQUIFIT

SILVERSOX™

Directions:

1. Cut to desired length (use sharp scissors to prevent fraying).
2. Scrunch material and roll over hoof, then pull down leg in the direction of hair growth.
Tip for easy application: Slide a plastic bag over the hoof and leg and then pull SilverSox over the bag. Once in place, slide the bag out.
3. Place SilverSox in desired location and apply boot or wrap. SilverSox should extend above the top of boot or wrap to stay secure.
4. For heel coverage, extend SilverSox to hoof and secure with duct tape (avoiding hair).
5. Remove after use/work. SilverSox are reusable for 1-3 uses and are machine washable on delicate cycle and low heat.

Note: If any increased reaction occurs around irritation, discontinue use and consult your veterinarian.

EquiFit, inc.

BETTER FIT. FOR A BETTER FINISH.
WWW.EQUIFIT.NET
877.437.8434



SilverSox combine all the benefits of compression with the power of X-STATIC® Antimicrobial Technology. Made with 99.9% pure silver, X-STATIC® enhances SilverSox with permanent anti-odor and antimicrobial performance. It naturally inhibits the growth of bacteria in apparel and textiles, keeping odors in check and gear smelling fresher. X-STATIC® also has natural temperature regulating properties, reflecting heat back to keep warm and evaporating moisture faster to keep your horse's leg cool.

2 Yards · Made in the USA
