

FOR ALL EQUINE ATHLETES AT ALL LEVELS OF TRAINING AND PERFORMANCE

How to apply FLAIR® Nasal Strips



Clean & Dry

1 **IMPORTANT:** Clean and dry the muzzle thoroughly. (Oil, dirt and sweat will reduce adhesion.) Apply 30 minutes before exercising horse.



Expose Adhesive and Place Strip As Shown

2 Immediately before applying the strip, remove the 3 part protective back liner from the underside of the strip. **DO NOT TOUCH THE ADHESIVE SURFACE.** While holding the Handling Card, place bottom edge of strip 1.5" above the top of each nostril. Press down to adhere strip.



Remove Handling Card

3 After removing Handling Card, rub strip to secure adhesive.



4 This is how the FLAIR Nasal Strip should look when positioned correctly.

A helpful pre-measured positioning guide is included inside. Scan the QR code on the right with your smart phone for a "How to apply" video.



How FLAIR® Nasal Strips work

Without FLAIR Nasal Strips
Nasal passages collapse without extra support, restricting airflow.

With FLAIR Nasal Strips
Spring-like action gently holds nasal passages open to improve breathing.



When horses exercise heavily, their nasal passages collapse. By preventing this collapse, FLAIR Nasal Strips are clinically proven to:

- Reduce lung stress and bleeding (EIPH)
- Reduce fatigue
- Conserve energy
- Speed recovery

See the science at our website: www.flairstrips.com



PREVENT PROTECT PERFORM™

For information call: 1-888-68-FLAIR or 763-972-9056

TO REMOVE: Peel gently, but firmly from the top corner toward the middle. Do not remove by pulling up or down. Adhesive may cause slight loss of hair as experienced with normal shedding.

CAUTION: For external use only. Do not apply to sores, broken or irritated skin. Intended exclusively for horses. Store at room temperature; avoid extreme heat or cold.

Manufactured by Flair, LLC, Delano, MN 55328. US Patents: 5913873, 6017357, 6203560, 6352548, 7175645 and other patents pending. Made in USA © 2012 Flair LLC