

Legacy

SYSTEM FITTING GUIDE



STEP 1

Fit the splint bar to the inside of the leg and line the back bottom edge of the boot clearly with the base of the fetlock/ankle.



STEP 2

While holding the splint pad in place, firmly secure the top two loop closures around the outside of the boot.



STEP 3

Wrap the boot's cradle around the base of the fetlock so that the 2 fitting darts on the cradle line up evenly at the back of the fetlock.



STEP 4

Secure the hook-and-loop closures to the front of the boot.



NOTES:

The boot should be snug and clean fitting. The boot cradle is contoured to fit comfortably around the base of the fetlock.

Classic  Equine®

